

Before You Begin:

(Be in the bathroom or have supplies ready.)

CAUTION

**Do not disconnect the tubing
From the reservoir at any time.**

To Empty:

Unplug drainage plug from the emptying port, hold the reservoir upside down over a measuring container and squeeze the reservoir to empty the collected fluid. When the reservoir is emptied of all fluid, you will need to reactivate it.

To Reactivate:

Squeeze to compress the reservoir as completely as possible.

With the reservoir compressed, insert the drainage plug into the emptying port as far as possible and release the reservoir. The reservoir should expand gradually as it fills with fluid. If the reservoir does not expand gradually and collect fluid, contact your physician.

Attach the reservoir (using the plastic loop) to your clothing as directed by your physician. Place the reservoir below the drain site to maintain adequate drainage. Be certain to empty the reservoir before the fluid collected reaches the 100cc mark.

To Summarize:

1. Observe and record the amount of fluid collected on the Drainage Volume Chart.
2. Wash hands thoroughly before handling the reservoir.
3. Holding the reservoir over the measuring container, unplug the drainage plug and squeeze the reservoir until empty.
4. Reactivate the drainage system by compressing the reservoir and inserting the drainage plug as far as possible.
5. Flush the drainage fluid down the toilet after measuring.
6. Wash hands thoroughly.

Caution:

- Do not disconnect, kink or puncture the tubing that is connected to the reservoir.
- Do not let the reservoir completely fill because the drainage will stop.
- Do not disconnect the tubing from the reservoir.

If any of these problems occur, contact your physician.

